

Massage Chair

Manual



Printed in China

CONTENTS



Thank you for purchasing our massager chair.
Please read the owner's manual(operating manual) before using.
Please keep the manual together with the warranty card properly.

Safety precautions	2-5
Main function	6
Components & Descriptions	7
Before using	8-10
Adjusting massage chair	11-12
Program and features	13
Usage	14-15
Troubleshooting	16
Regular maintenancae for safety	17
Storage and care	18
Product specification	19




This product is designed for residential use. Do not use it for other purposes.
Please take out the power plug from the power socket immediately when fire or other
emergency happens.

SAFETY PRECAUTIONS






- Please adhere to all precautions for correct operation.
- This manual contains important information about how to use the chair safely and correctly, avoiding possible damage to property or injury to persons using the chair and others.
- To clearly indicate the degrees of danger and damage, symbols “warning” and “caution” should shown below to highlight important safety instructions.




 WARNING	This action could result in serious injury or death.
 CAUTION	This action could result in minor personal injury or damage to property. ※ Damage to property is damage to house, property, livestock and pets.







[Symbols]

	Warning or Caution (This symbol means warning or caution before use.)
	⊘ Prohibited (The symbol on the left means “Do not disassemble”).
	Required Action



- Remember to keep the manual in an easily accessible place.

 WARNING	
<p>If you find any abnormal condition, please stop operation immediately.</p> <p>Electric shock or fire may result in if the chair is operated under such circumstances. Please turn off the main power switch on the chair and unplug the chair from the wall outlet. Contact the vendor for inspection or repairing.</p>	
	<ul style="list-style-type: none"> ● If there is a threat of lightning, abnormal noise (any abnormality), turn off the main power switch. For the safety, contact the vendor for inspection. Do not try to repair any malfunction of the chair.
	<ul style="list-style-type: none"> ● Any moisture into the chair. ● Any obstacle into the chair. ● Any drooping down, tipping over or broken on the out housing.
 CAUTION	
 REQUIRED ACTION	<ul style="list-style-type: none"> ● If you feel uncomfortable during operation, please stop using immediately. Consult with your doctors. ● Use the chair only when it is placed on a flat surface. ● Before operation, make sure the switch is in order. Or it will result in injury and accident.






 WARNING	
 PROHIBITED	<p>The following persons must not use the chair.</p> <p>Persons who have thrombosis, serve aneurism, acute varicosity, skin infection, subcutaneous tissue inflammation.</p>
 REQUIRED ACTION	<p>Persons with any of the following conditions should seek medical advice before using.</p> <ol style="list-style-type: none"> 1. People who have electronic medical devices easily susceptible to electromagnetic interference. 2. People with malignant tumors. 3. People with serious heart disease. 4. Woman who is in menstrual period or pregnant. 5. Sensory disturbance caused by peripheral circulatory disorder due to diabetes. 6. People with sensory disturbance. 7. Persons being treated for personal injury. 8. High fever (38 °C or higher) such as acute inflammation, high blood pressure or chills. 9. Persons with sickness or under bed rest orders. 10. Osteoporosis, spinal fractures and acute pain. 11. Inflammation due to sprains and muscle strain. <p>It is not recommended for those who are in uncomfortable and sickness.</p> <p>Keep massage to no more than 15 minutes in length. Do not continue to massage an area specific of any particular of your body. Please stop for at least 30 minutes before reuse. Continue using for extend periods of time can result in over stimulation on muscles and nervous.</p>




 WARNING	
 FORBIDDEN WET HAND	<p>Do not use the chair when your body is wet. It could result in electric shock or malfunction.</p>
 GUARD AGAINST DAMP	<p>Do not spill water or other liquid on the chair. Do not place it in the humid environment like bathroom. It may result in electric shock or malfunction.</p>
 REQUIRED ACTION	<p>Make sure no one especially pets in the way when operation.</p>
 DO NOT DISASSEMBLE	<p>Do not try to disassemble or repair the chair, power cord and controller. It may result in fire and malfunction.</p>
 PROHIBITED	<p>Do not destroy the power cord.</p> <p>The power cord should not be broken or damaged. Do not bend, stretch, twist or wrap the power cord. Do not hang up objects on the power cord. The broken cord will result in electric shock, short circuit or fire.</p>



SAFETY PRECAUTIONS

 WARNING	
 PROHIBITED	This product is not intended for use by children and persons with reduced mental capabilities. Keep children and persons with reduced mental capabilities away from the chair during operation.
	Do not use the product with anything other than an AC220V power source. Do not wrap the power cord around the main body or it will result in electric shock, short circuit and fire.
	Do not let children sit and play on the product or it will result in injury and malfunction.
	During the operation, do not put your fingers into the gap between the housing and cushions or it will result in injury and accident.
	Do not use the chair to massage head or stomach, or it may result in nausea or injury
	Do not use the chair unclothed or in close proximity to loose clothing or jewelry, it may injury to you.
	Should the fabric cover be broken or removed, please do not use the product.

If you feel uncomfortable during use, please stop the product immediately and consult with your doctors.

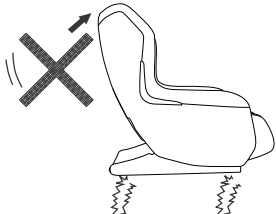 CAUTION	
 REQUIRED ACTION	<ul style="list-style-type: none"> In the event of power failure, switch off the main switch and disconnect the power plug to prevent any sudden operation if the power is restored unexpectedly.
 FORBIDDEN WET HAND	<ul style="list-style-type: none"> Do not touch outlet, plug and power cord with wet hands.
	<ul style="list-style-type: none"> After use or in the event of power failure, please disconnect the power plug to prevent electric leakage, electric shock and accident . In the event of cleaning the product, please disconnect the power plug to prevent electric shock and fire. When disconnecting the power plug from the socket, be sure to hold the plug instead of pull over the power cord. Or it will result in electric shock, short circuit and fire. Please check power plug and power cord as schedule.
 PROHIBITED	<ul style="list-style-type: none"> Do not use the product immediately after drinking and dinner. Or it will result in uncomfortable. Do not hangup heavy objects with power cord and control wire. Do not drop the controller or it will result in electric shock and fire. Do not use the product near the fire or heater device. Do not smoke while you are using. Do not place the product on the floor where there is underground heating or other heating device. Or it will result in fire.


 CAUTION	
 PROHIBITED	<ul style="list-style-type: none"> Do not pull or crash the fabric cover with sharp implement, like knife. Or it will result in injury and malfunction. Do not unduly twist stretch the power cord or it will result in fire. Do not drop off the controller, avoid the controller from external impacting. Or it will result in malfunction. Should any broken on the power cord and power plug, or any loosen on the outlet, please do not use the product, or it will result in electric shock, fire and accident. Do not put your fingers into the gap of the chair when reclining the chair. Do not force overloading on each function courses or it will result in malfunction and injury. Do not use the product when you fall in asleep. Use with unconsciousness will result in injury and uncomfortable. Do not unduly pull over the product from the wooden floor, carpet or other place which is easily scratched. Please carry and move the chair carefully to avoid scratching the floor.
	

 CAUTION	
 PROHIBITED	This product is designed for home use and not for commercial purposes.
	Do not operate the chair covered by a blanket. Excessive heating will occur and cause fire and accident.
	Make sure the well connection between the plug and the outlet. Wipe dust away from them, or it will result in electric shock, short circuit or fire.
	Do not stand or tread on the product. Do not force overloading on the product, or it will result in injury, malfunction and accident.
	Do not use the chair near fire resource or flammable gas, or it will result in product distortion, malfunction and fire accident.
	Do not use the product where aerosol spray products are being used or where oxygen is being administered. It will result in product distortion, malfunction and fire accident.
	Do not clean or spray the product with abrasive cleaning detergent, alcohol or insecticide, or it will result in product distortion, electric shock and fire accident.
	Please keep children and persons with reduced mental capabilities away from the running product, or it will result in injury.
	This product is not intended for use by persons with reduced mental capabilities or under bed rest orders. It may result in personal uncomfortable.


MAJOR FEATURES

- 1 More fashionable appearance**
Excellent colour matching and exquisite workmanship make a great massage experience
- 2 More comfortable body feel**
Smart body scan system, 130cm SL-track serves to reach thigh, conform to the spines natural shape
- 3 More smart massage**
One-touch Zero Gravity provides a weightless felling to relieve tension, stress and muscle discomfort
- 4 Smart program makes easy operation**
Button designed at two side of the chair makes easy operation
- 5 Zero wall technology**
Zero wall technology allows the chair to be positioned within 10 cm of a wall
- 6 More professional massage technology**
Tapping, kneading, shiatsu massage
- 7 Bluetooth music speaker**
Bluetooth technology, stereo music speaker

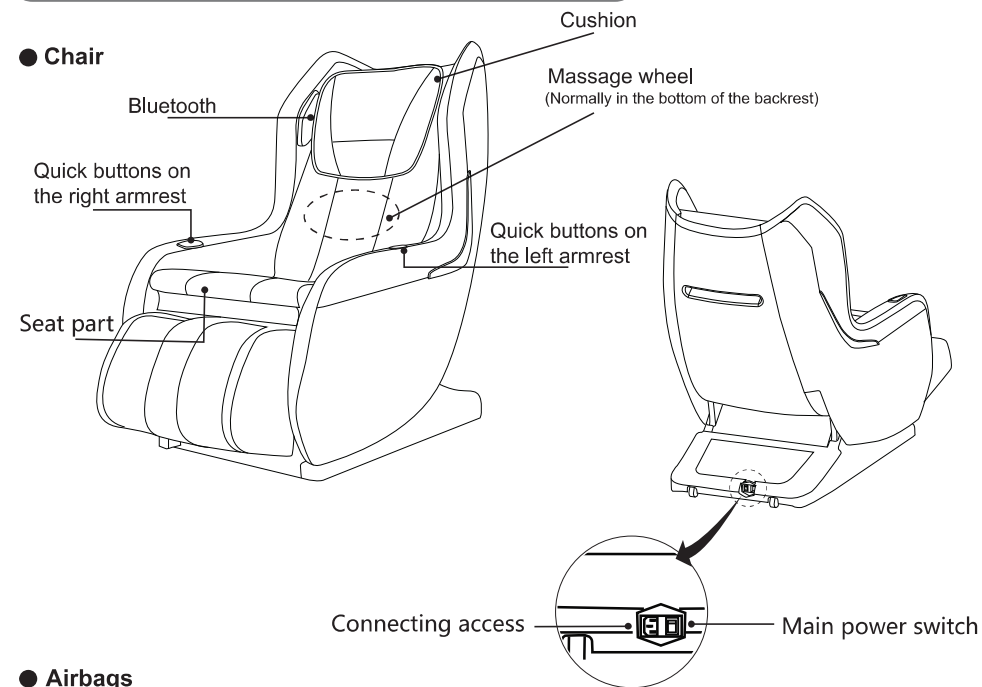

COMPONENTS AND DESCRIPTION

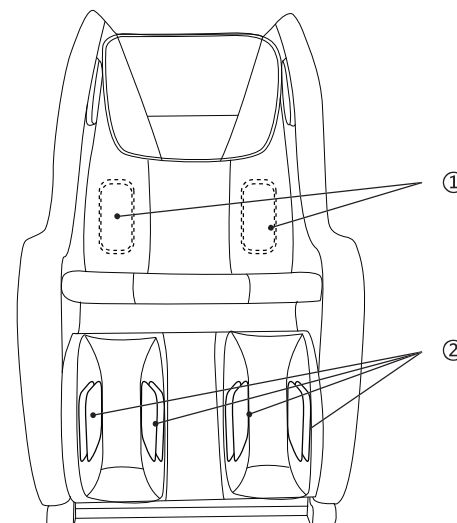

● Airbags



- ① Waist massage airbags X 4
- ② Leg massage airbags X 8


 WARNING	<ul style="list-style-type: none"> ● Please do not spray water or other liquid on the chair. ● Do not operate the chair with wet hand.
 CAUTION	<ul style="list-style-type: none"> ● Do not sit on the controller, do not drop or step on the controller. Do not pull over the wire . ● Do not press the button with sharp object or it will result in damage.

BEFORE USE


- This chair is packaged with the backrest flatwise. Before use, please raise up the backrest .
- When the temperature is too low to run the machine, please increase the ambient temperature gradually, do not increase the temperature in sudden. Please move the chair to the warm place and keep on waiting for 1 hour before starting.
→ Or it will result in moisture of parts and abnormal operation.

Where to place the chair

Please make sure there is 25cm in front of and 10cm behind of chair to prevent the chair from scratching on the wall or other objects when reclining.


 CAUTION	<ul style="list-style-type: none"> ● Do not use the chair in the bathroom, sauna room or other places with high humidity. ● Please place the chair on the flat floor ● Do not use the chair in places with sundries and dust or with high ambient temperature (above 40℃) ● Do not put the chair in places with direct sunlight or near heating equipment with high temperature. ● Keep the chair away from the cigarette or flame.
--	--

- Lay a mat under the chair to prevent damages to the floor and for reduction of operation noise. (Do not lay electric blanket)
 - Due to the fact that the chair is heavy, make sure that the floor will not be damaged when placing it on the floor.
 - Place put the chair 1 meter away from the TV, radio and other audio or video devices, otherwise the interruption noise may occur.
- The chair is off limit to users with weight over 100kgs.

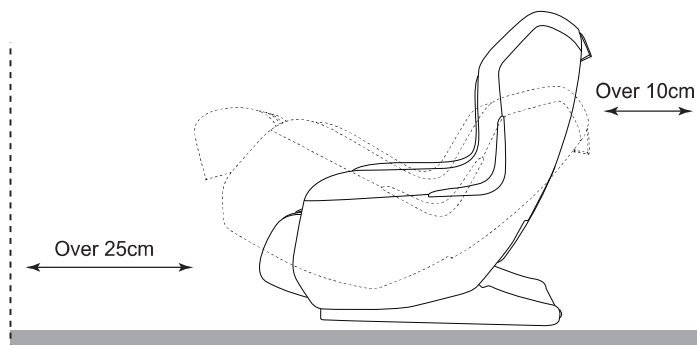

Confirm the articles inside the package

Make sure there are massage chair and accessories in the package.

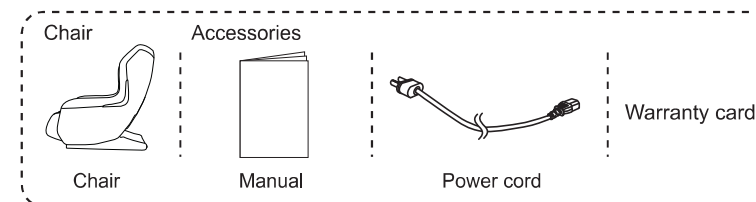



How to move

Please switch off the power before moving the chair and pull the power cord out of the outlet.

- Move with casters


Please press down the chair backward until it sits on the rear wheels, then move the chair forward or backward.

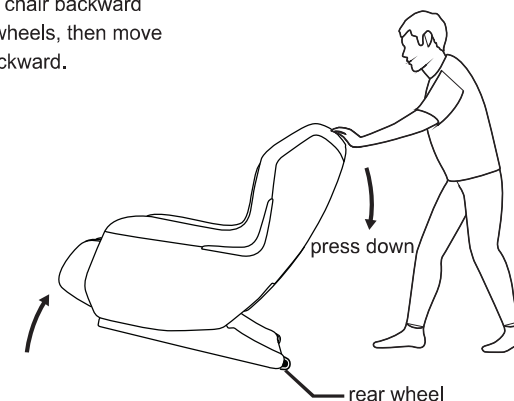


 CAUTION	<ul style="list-style-type: none"> ● The chair cannot be moved if the passage is less than 72 cm in width. ● Contact the dealer in case you cannot move the chair normally. ● Due to the heavy weight of the chair, make sure that it will not damage your waist or other parts of your body during moving. ● The controller is connected on the chair. Make sure to avoid its dropping off during moving ● Do not move the chair with person loaded. ● Please move the chair after unplugging. → Otherwise it may cause damages to the power plug, cause fire disaster or electric shock.
--	--

How to switch on the power

① Plug the power cord to the switch socket on the back of the chair, and the other end to the power outlet.

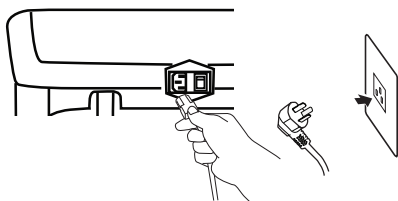

② Turn on the main power switch in the back of the chair.

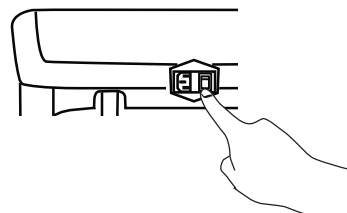

※ If the rollers gets stuck, turn on the switch and press the ON/OFF button, the roller will be back to its initial position.(the lower back of the chair)

⚠ WARNING

- Do not use external transformer or it will result in accident or fire.

⚠ CAUTION

- Be sure the main switch is "OFF" before plug in the chair.
- Be sure the chair is well plugged in.
- Before use, please read the manual carefully and set the chair in light of the requirement to ensure the normal operation.


Confirm the fabric

Before use, please confirm whether the fabric is damaged or not.


① Open zipper and back cushion from one side

② Put hand between the back cushion and backrest to confirm if the fabric is damaged or not.

Open from this side

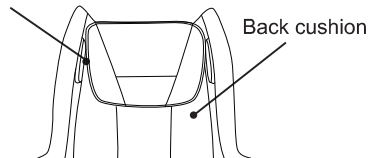

Fabric

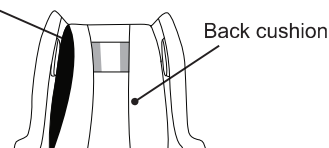

⚠ CAUTION

- Before using the chair, confirm that whether there is damage in the leather and fabric or on other parts. In case of damage, even the smallest tearing n the chair, please stop using immediately, turn off the power, unplug the chair and contact the customerservice center. Do not remove the cushion or broken fabric, or it will result in safety hazard.

ADJUST THE CHAIR

Sitting in the Chair


Under general circumstances, the rollers are laid folded at the lower waist of the backrest. Please ensure the position of the rollers for safety. If the rollers are laid otherwise, restart the chair, keep on waiting for the parts return to its initial condition and sit in the chair.

⚠ CAUTION

- Do not stand in the footrest when raising the footrest.
→The overload on the footrest will result in accident or reclining chair will cause personal injury.
- Raise the chair back up before using. Enjoy the massage from light massage course. Adjust the angel of your reclining angel according to your body figure.

Adjust the pillow

- If massage intensity of neck needs to be strengthened,please fold the pillow.

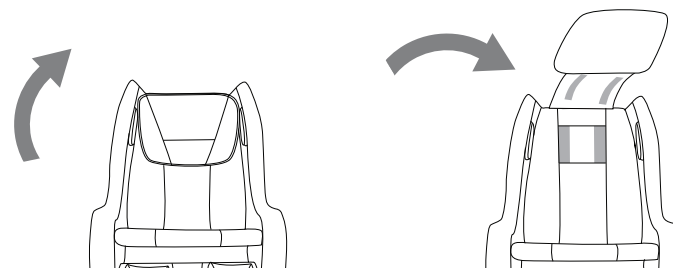


⚠ CAUTION


When pillow is folded,massage intensity of neck is strengthened,please keep massage time within 15 minutes. There is heavy object in the balancing bag of pillow,do not play with it,otherwise it may cause injury.Keep children away from this place

Adjust the backrest and the footrest

- Control the angle of backrest by pressing the quick buttons on the left armrest.

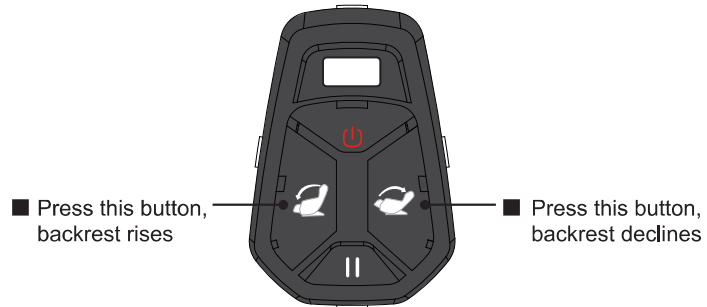


Raise and decline are the continuous movement of the chair backrest and the legrest
(Stop tilting means both the backrest and the legrest)

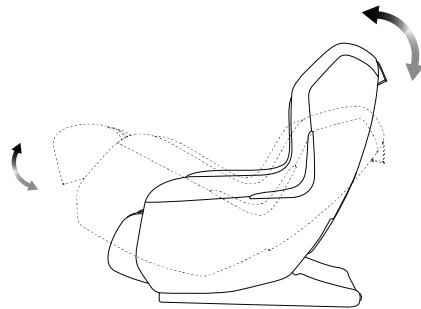

WARNING When adjusting angle, please make sure there is no person (Especially children) or pet around. Besides, there must be at least 10cm space at the front or back of the chair, and there is no foreign matter in the way when rotating.


Program and features

Quick buttons on the left armrest

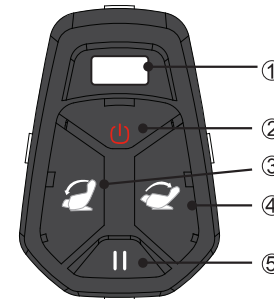

①	USB	USB charging
②	Power	ON/OFF
③	Backrest up	Press this button to raise the backrest
④	Backrest down	Press this button to decline the backrest
⑤	Pause	Press this button, the chair stops massage. Press this button again, the chair go on with massage.


Quick buttons on the right armrest


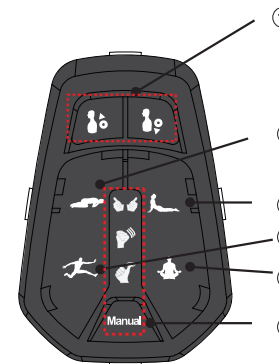


①	Neck/shoulder adjustment		Get a comfortable position by adjusting the neck/shoulder massage wheel
②	Sleeping		Calm mind
③	Decompress		Relieve fatigue
④	Vitality		Body rejuvenation
⑤	Healthcare		Relaxation
⑥	Manual		Press this button to choose 3 massage methods, kneading, shiatsu, tapping

Usage

1 Connect the power

- Insert power cord into outlet.
- Turn on the main power, the USB indicator light on the left armrest will work, on-off key () indicator light flashes.

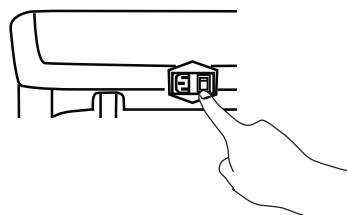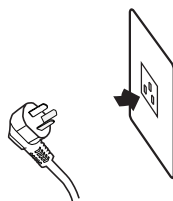

2 Press the buttons

- 1 Press the  button, massage chair on standby state.



- 2 Four AUTO programmes: healthcare, decompress, vitality, sleeping can be changed to each other after press the key on the controller, meanwhile the indicator light will on


- The massage time of healthcare programme is 6 minutes
- The massage time of decompress programme is 30 minutes
- The massage time of vitality and sleeping are both 20 minutes






- Different modes can be changed to others when the chair is working
- The massage movement moves the whole body under the 4 AUTO programmes

- 3 There are 5 massage methods ( kneading,  tapping,  shiatsu,  kneading+tapping,  tapping+shiatsu) in manual programmes.

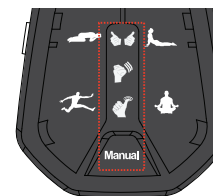

- Different massage methods can be selected one by one as below

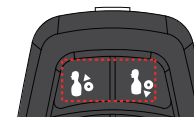

- 5 massage methods work over the whole programme, user can adjust the movement to the best position by pressing the  / 
- The adjustment just can be operated under manual programme



- 4 The angle of the massage chair lying down can be adjusted

- The backrest can be raised and declined by pressing  / 
- The angle can be adjusted after body scanning
- Backrest and legrest are linkage statement



- 5 Please press pause button when you feel uncomfortable or some emergency happens during the massage

The massage ball stops at the position when press the pause button and the indicator light on the right armrest flashes. After press the pause button again, the chair continues working




3 After use

Press the power button when you want to finish massaging or stop massaging midway, the chair will stop working

- After the massage, turn off the power button, pull out the plug which is good for energy-saving.

TROUBLE SHOOTING

If there is still a problem after trying the solutions given in this troubleshooting guide, stop using the chair immediately to avoid injury or incident. Unplug the chair from the outlet, contact the vendor or your nearest service center for assistance.

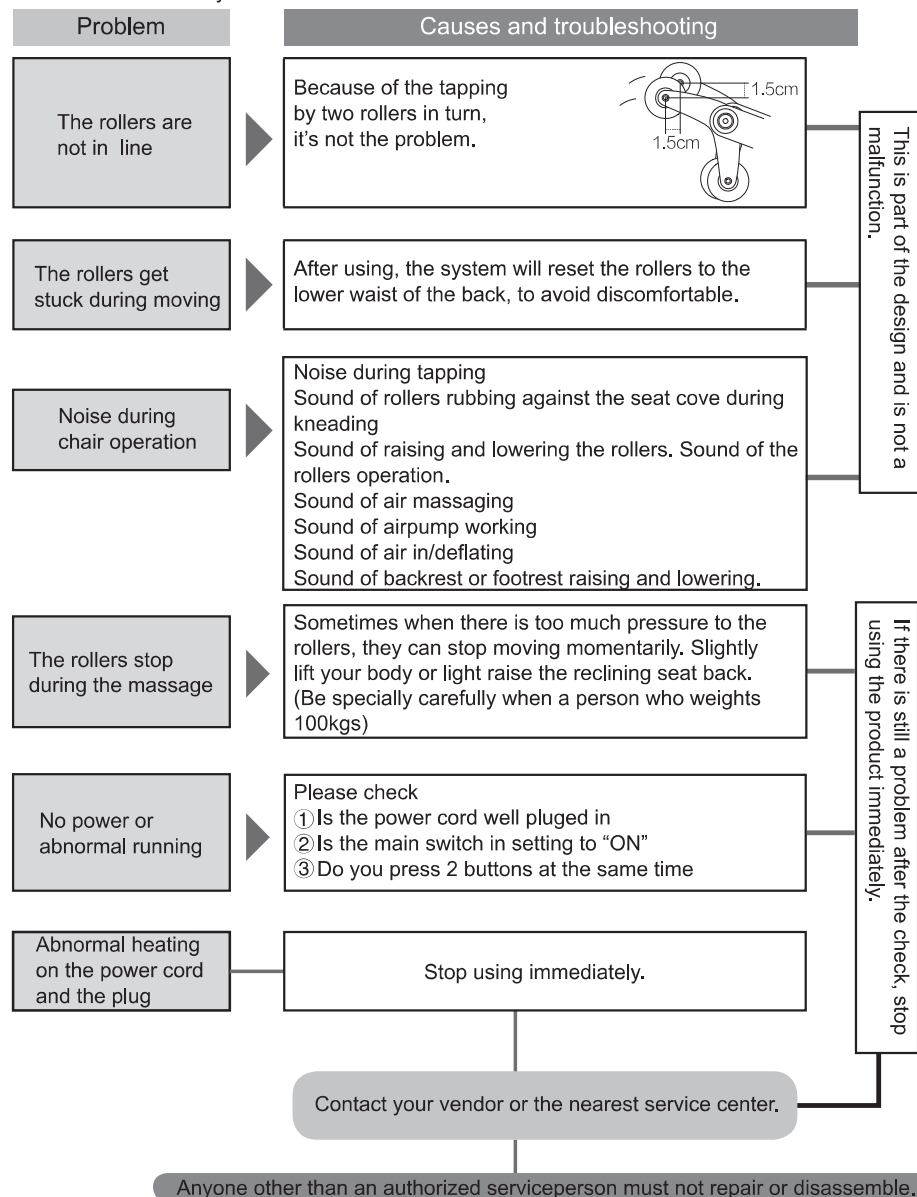

Anyone other than an authorized serviceperson must not repair or disassemble.

REGULAR MAINTENANCE FOR SAFETY

Regular maintenance will preserve the life of your chair.

Have you experienced any of these problems?

- The chair can not start with the main power switch set to ON.
- The power cord has been cracked or scratched.
- The power cord or plug becomes abnormally hot.
- The chair produces a burnt smell, abnormal sound or vibration.
- The auto timer function does not work in some cases.
- Other failures.

If you find anything listed on the left, stop using and unplug the power cord from the outlet to prevent failure or an accident. Be sure to contact the vendor or our service center for maintenance.

To use safely

The chair fabric is a consumable part with service life of 3 years.

(Continuous of use: 2times a day with each cycle 15 minutes).

For safe use, periodic inspection is recommended before the end of its service life.

- ※ Durable years vary depends on conditions of use.
- ※ After 5 years of use, a professional inspection is recommended even if there are no symbols or problems. For inspection cost please contact the vendor.

CARE AND STORAGE

STORAGE

- Loose the hasp, fold the chair back into the chair.
- If the chair is left unused for a while, cove the chair to prevent from dust.
- Pets sometimes chew on power cord, try to keep the pets away from the chair even when it is stored.

CAUTION

- Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.

CARE AND MAINTENANCE

CAUTION

- When cleaning the chair, be sure to unplug the chair from the outlet. Do not plug or unplug the chair with wet hands.

[Caring for the back cushion and covers]

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Then thoroughly wiring it out before cleaning off any spots or grime.

CAUTION

- The fabric color can change if the chair is in contact with vinyl covers or wall coverings for a prolonged period of time.

[Caring for other plastic parts]

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Thoroughly wiring it out. Clean off any spots or grime. Then wipe with a dry cloth so that no detergent residue remains.

CAUTION

- Do not use alcohol based or other household cleaning products or abrasive because they can scratch or crack the finish or cause color to fade.

[Care for the controller]

Wipe dirt off the controller with dry cloth.

CAUTION

- Never use a moistened or wet cloth. Or it may result in accident.

SPECIFICATION

Voltage	220V~ 50/60Hz
Power	100W
Time	30 min
Size	Approx 72*125(148)*117(91) cm (W*D*H) The length and height figures in () represent the dimensions when fully reclined.
Weight	About 70 Kg
Cover	PU leather、 Fabric
Accessories	Power cord, Manual , Warranty card

Chair Size

